

ONE TEST, TWO RESULTS

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™

Simple and accurate performance

- Simultaneous detection and differentiation of *Giardia* and *Cryptosporidium* using one test
- Less labor intensive than traditional methods
- Non-subjective results in 30 minutes without instrumentation



GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™

Product Order Information

Product Name	Cat. No.	Sample Type	Time to Result	Format
GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™	T30407	Fresh and frozen fecal sample (unpreserved), 10% formalin, SAF, Cary Blair, C&S	< 30 minutes	Rapid EIA

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™


Assay Protocol

1

Dilute fecal specimen

For fresh or frozen specimens, add 25 µL of specimen to tube containing 500 µL of *Diluent* and 1 drop of *Conjugate*. Mix thoroughly.


For preserved specimens, add 100 µL of specimen to tube containing 400 µL *Diluent* and 1 drop *Conjugate*. Mix thoroughly.

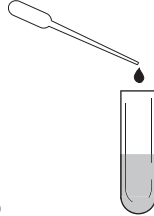

2

Add sample


Transfer 500 µL of diluted sample-conjugate mixture to *Sample Well*. Incubate at room temperature for 15 minutes.

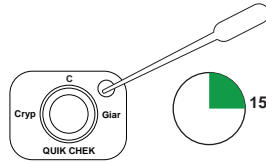

3

Wash Membrane Device


Add 300 µL *Wash Buffer* to *Reaction Window*. Allow to completely absorb.

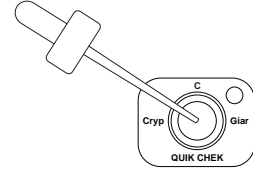

4


Add Substrate reagent

Add 2 drops *Substrate* to *Reaction Window*. Incubate at room temperature. Read at 10 minutes after addition of *Substrate**.

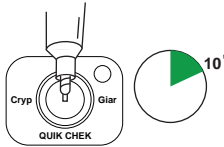


*A positive result may be interpreted at any time during the 10-minute incubation. A test cannot be ruled as negative or invalid until after the 10-minute incubation.


Easy to Read Results

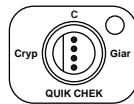


Positive Crypto.
Result

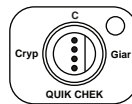

Positive Giardia
Result

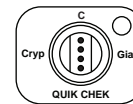

Positive Crypto. and
Positive Giardia Result

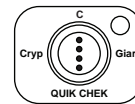

Negative Result

If there are no dots at C (control) the test is invalid. See package insert for additional interpretation of results.

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK™ Package Insert

Distributed by

keydiagnostics

T: 02 8212 4074 F: 02 9423 6992
info@keydiagnostics.com.au
www.keydiagnostics.com.au
PO Box 1038, Gympie, NSW, 2227

Developed and manufactured by

TECHLAB®

PROVEN DIAGNOSTIC PERFORMANCE

www.techlab.com

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.

© 2021 TECHLAB, Inc. All rights reserved.

GIARDIA/CRYPTOSPORIDIUM QUIK CHEK, the TECHLAB Logo, and TECHLAB are trademarks of TECHLAB, Inc. All other trademarks referenced are trademarks of their respective owners. PN 832021006