

RELIABLE RESULTS
***H. pylori* Testing**

Actionable results you can trust

- Follow the most recent ACG Guidelines by using a stool antigen test to detect *H. pylori* infection
- Identify active *H. pylori* infection before and after treatment with a non-invasive test
- Obtain results quickly to improve patient care
- Reduce sample rejection with flexible sample transport conditions



***H. pylori* Test Kits**

Product Order Information				
Product Name	Cat. No.	Sample Type	Time to Result	Format
<i>H. PYLORI QUIK CHEK™</i>	30925	Fresh, frozen, transport media (Cary Blair or C&S)	30 minutes	Rapid EIA
<i>H. PYLORI CHEK™</i>	T5051	Fresh, frozen, transport media (Cary Blair or C&S)	~ 60 minutes	Automatable 96-well plate

H. PYLORI QUIK CHEK™ Performance

N = 122	CRM* Positive	CRM Negative
H. PYLORI QUIK CHEK™ Positive	32	0
H. PYLORI QUIK CHEK™ Negative	1**	89

	95% Confidence Limits	
Sensitivity	97.0%	84.7% - 99.5%
Specificity	100%	95.9% - 100%

* CRM = Composite Reference Method consisting of urease testing and histology of biopsy samples.

** Additional testing with a US FDA-cleared *H. pylori* stool antigen test provided an antigen-negative result


H. PYLORI CHEK™ Performance

N = 109	CRM Positive	CRM Negative
H. PYLORI QUIK CHEK™ Positive	32	3***
H. PYLORI QUIK CHEK™ Negative	0	74

	95% Confidence Limits	
Sensitivity	100.0%	89.3% - 98.9%
Specificity	96.1%	89.2% - 98.7%

***All three specimens tested positive initially by the H. PYLORI CHEK™ test, but negative upon re-testing with the H. PYLORI CHEK™ test.

Chey, WD, Leontiadis GI, Howden CW, Moss SF. 2017 ACG Clinical Guidelines: Treatment of *Helicobacter pylori* Infection. Am J Gastroenterol 12:212-238.
H. PYLORI QUIK CHEK™ Package Insert and H. PYLORI CHEK™ Package Insert

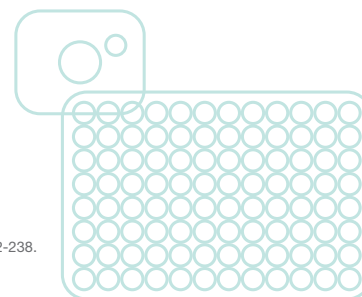

Distributed by

keydiagnostics

T: 02 8212 4074 F: 02 9423 6992
info@keydiagnostics.com.au
www.keydiagnostics.com.au
PO Box 1038, Gymea, NSW, 2227

Developed and manufactured by



PROVEN DIAGNOSTIC PERFORMANCE

www.techlab.com

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.

© 2021 TECHLAB, Inc. All rights reserved.

H. PYLORI CHEK, H. PYLORI QUIK CHEK, the TECHLAB Logo, and TECHLAB are trademarks of TECHLAB, Inc. All other trademarks referenced are trademarks of their respective owners. PN 832021012