

DISTINCT RESULTS

E. HISTOLYTICA QUIK CHEK™

Provide quick, straightforward results for better patient care

- Specifically detect *E. histolytica* without cross-reacting with non-pathogenic *E. dispar*
- Avoid inappropriate therapy and unnecessary antibiotic treatment
- Report results in minutes, not days



E. HISTOLYTICA QUIK CHEK™


Product Order Information

Product Name	Cat. No.	Sample Type	Time to Result	Format
<i>E. HISTOLYTICA QUIK CHEK™</i>	T30409	Fresh and frozen fecal sample (unpreserved) Cary Blair, C&S	< 30 minutes	Rapid EIA


E. HISTOLYTICA QUIK CHEK™

Assay Protocol


- 1** Add to a test tube:
- 500 μ L *Diluent*
 - 1 drop *Conjugate*
 - 25 μ L (1st graduation) of specimen
 - Mix thoroughly

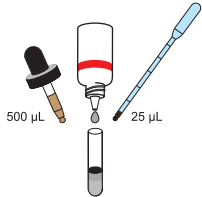


- 2** Transfer 500 μ L (highest graduation) from tube to small *Sample Well*. Keep the cassette at room temperature and wait 15 minutes.

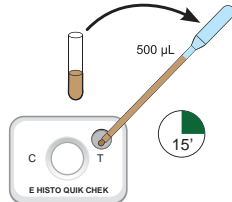

- 3** Add 300 μ L *Wash Buffer* to large *Reaction Window*. Allow to completely absorb.

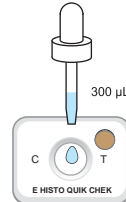


- 4** Add 2 drops *Substrate* to large *Reaction Window*. Keep the cassette at room temperature and wait 10 minutes. Read results.

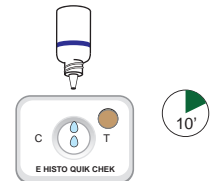


Easy-to-read Results

- Results in 30 minutes without instrumentation
- Uses the popular TECHLAB® QUIK CHEK™ format
- Simple and clear interpretation of results

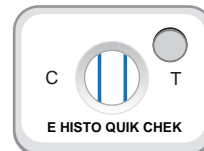

Positive Result

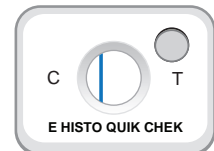


Negative Result

E. HISTOLYTICA QUIK CHEK Package Insert
Haque, R. et al. 2003. Current Concepts Amebiasis. N Engl J Med. 348:1565-73.

Distributed by

keydiagnostics

T: 02 8212 4074 F: 02 9423 6992
info@keydiagnostics.com.au
www.keydiagnostics.com.au
PO Box 1038, Gympie, NSW, 2227

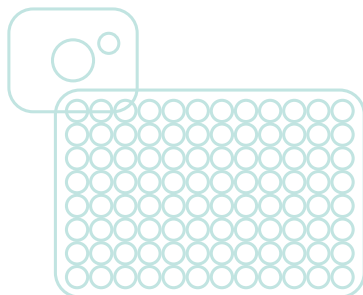

Developed and manufactured by

TECHLAB®

PROVEN DIAGNOSTIC PERFORMANCE

www.techlab.com

TECHLAB, Inc. is a leading developer and manufacturer of infectious disease diagnostics products worldwide. Since 1989, the company has offered a quality portfolio of diagnostic tests which now includes assays for *C. difficile*, *Campylobacter*, Shiga toxin producing *E. coli*, *H. pylori*, protozoan parasites, and biomarkers of intestinal inflammation.

© 2021 TECHLAB, Inc. All rights reserved.

E. HISTOLYTICA QUIK CHEK, the TECHLAB Logo, and TECHLAB are trademarks of TECHLAB, Inc. All other trademarks referenced are trademarks of their respective owners. PN 832021009